NObreath[®]

Quick Start Guide

*Aids in the diagnosis & management of
asthma, one breath at a time.*

CE
2797

bedfont[®]
est. 1976

Contents

English (EN).....	Page 4,5
Français (FR).....	Page 6,7
Deutsch (DE).....	Page 8,9
Español (ES).....	Page 10,11
Italiano (IT).....	Page 12,13
Português (PT).....	Page 14,15
Nederlands (NL).....	Page 16,17
Polski (PL).....	Page 18,19
Svenska (SE).....	Page 20,21
Suomi (FI).....	Page 22,23
日本(JP).....	Page 24,25
Русский (RU).....	Page 26,27
český (CZ).....	Page 28,29
Ελληνικά (GR).....	Page 30,31

Features

FeNO testing
without limits*

Easy to use

Portable

Adult and child
FeNO testing

PC Software

Low-cost

Low
maintenance

Bluetooth[®]
Technology

* Subject to correct use, maintenance and service. Tested up to 29,000 tests.

PLEASE CHARGE THE DEVICE IMMEDIATELY!

The NObreath[®] should be charged for a minimum of 24 hours before first use.

Taking a Breath Test

1. Press and hold the on/off button once to switch the device on and wait up to 60 seconds for the device to stabilise.
2. Attach a new mouthpiece to the Device and select either the adult  or child  breath test.
NOTE: To cancel the breath test, press the  icon.
3. As prompted onscreen by  or , inhale until prompted to exhale.
NOTE: Ensure the patient inhales through the mouth before exhaling through the mouthpiece.
4. When the exhale icons appear,  or , keep the device upright and blow gently into the mouthpiece. Make sure you are not covering the vent holes.
NOTE: The exhalation time is approximately 12 seconds for an adult and 10 for a child.
5. The onscreen flow meter will guide you on the exhalation rate: continue blowing until the green tick  appears onscreen to signify a successful test.
NOTE: If you blow outside of the exhalation guidelines, the test will beep, before signifying a fail, and a red cross  will appear. Touch  to retry the test or  to view the result.
6. Following the tick, a result will appear on the screen, in ppb.
7. You may save this result by pressing . Alternatively, you can return to the home screen by pressing .
8. If you choose to save your results you will be shown a list of patient profiles. Select a previously set profile to add your result to, or select an empty profile to set a new one. Set the profile by typing in a name or reference, and press .
9. Remove and dispose of the mouthpiece in accordance with local waste and disposal guidance.
10. If no further tests are required, press and hold the on/off button once to turn the NObreath® off and store on NObreath® Dock to charge unit.

NOTE: A PIN is required to save the reading to a patient profile, the default pin is 0000. It is highly recommended that you change the PIN to a 4 digit number that you will remember. If you have not entered your PIN in the last 30 minutes, you will be asked to enter it before you can access the patient profiles. If your PIN is entered incorrectly, you will be notified and the screen will refresh for you to re-enter your PIN. If you have forgotten your PIN, please contact Bedfont® or your local distributor to reset it.

For a full copy of the manual please contact your local distributor.

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© Bedfont® Scientific Limited 2024
Bedfont® Scientific Ltd.
Emergo Europe B.V.
Bedfont® Scientific Limited reserves the right to change or update this literature without prior notice.